
BRAUN

No touch + touch thermometer

BNT 400B

GB	English	2
DE	Deutsch	14
DK	Dansk	27
FI	Suomi	39
FR	Français	52
NL	Nederlands	66
NO	Norsk	79
SA	اللغة العربية	101
SE	Svenska	102

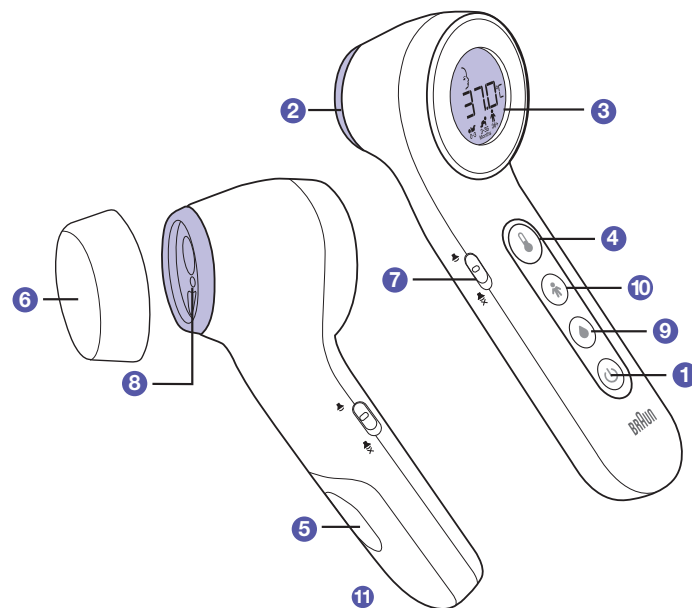This product is manufactured under license to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany.

Age Precision® is a registered trademark of Helen of Troy Limited.

 Kaz Europe Sàrl
Place Chauderon 18
CH-1003 Lausanne
Switzerland

© 2018 All rights reserved

www.hot-europe.com

Patents: www.kaz.com/patents/braun

Made and Printed
in China

BNT400BWE
A002836R0
22MAY18

Thank you for purchasing the No touch + touch thermometer (BNT400B). This thermometer is a high-quality product incorporating the latest technology and tested in accordance with international standards. With its unique technology, the product can provide a stable, accurate reading with each measurement. The thermometer performs a self-test every time it is switched on to always guarantee the accuracy of measurements. The Braun No touch + touch thermometer is intended for the intermittent determination of human body temperature on people of all ages in a "Touch" and "No Touch" mode, using the center of the forehead as the measurement site, in a home use environment.

Please read these instructions carefully before using this product and keep the instructions and the thermometer in a safe place.

Product description (See page 1)

- | | |
|---|--------------------------|
| 1 Power button | 2 Scanner |
| 3 LCD screen | 4 Temperature button |
| 5 Battery door | 6 Protective scanner cap |
| 7 Silent mode switch | 8 Guidance light |
| 9 Food and bath temperature mode | 10 Age Precision® button |
| 11 Fahrenheit to Celsius switch
(inside battery compartment) | |

WARNINGS AND PRECAUTIONS

Keep out of reach of children under 12 years.

NEVER use the thermometer for purposes other than those for which it has been intended. Please follow the general safety precautions when using on children.

NEVER immerse the thermometer into water or other liquids (not waterproof). For cleaning and disinfecting please follow the instructions in the "Care and cleaning" section.

DO NOT store this thermometer in temperature extremes below -25 °C or over 55 °C (below -13 °F or over 131 °F) or in excessive humidity (above 95% non-condensing relative humidity).

If thermometer is stored in a location that is cooler or warmer than where it will be used, allow it to acclimate to the room temperature for 10 minutes before taking a measurement.

DO NOT use the thermometer if there are signs of damage on the scanner or on the thermometer itself. If damaged, **DO NOT** attempt to repair the product.

NEVER insert a sharp object into the scanner area or any other open surface on the thermometer.

This thermometer consists of high-quality precision parts. **DO NOT** drop the instrument. Protect it from severe impact and shock. **DO NOT** twist the instrument or the measuring sensor.

This thermometer is intended for household use only.

Use of this thermometer is not intended as a substitute for consultation with your physician.

Temperature elevation may signal a serious illness, especially in neonates and infants, or in adults who are old, frail, or have a weakened immune system. Please seek professional advice immediately when a temperature elevation occurs on persons that are:

- Neonates and infants under 3 months (Consult your physician immediately if the temperature exceeds 37.4 °C [99.4 °F])

- Individuals over 60 years of age (Fever may be blunted or absent in older individuals)
- Individuals having diabetes mellitus or a weakened immune system (e.g., HIV positive, cancer chemotherapy, chronic immunosuppressant treatment, splenectomy)
- Individuals who are bedridden (e.g., nursing home patient, stroke, chronic illness, paraplegia, quadriplegia, surgical recovery)
- A transplant recipient (e.g., liver, heart, lung, kidney).

This thermometer is not intended for pre-term babies or small-for-gestational-age babies. Pre-term is defined as babies with a calendar age of less than 37 weeks. Small-for-gestational-age is defined as a baby born at 37 weeks or later, with a weight below the 10th percentile for babies of the same gestational age.

This thermometer is not intended to interpret hypothermic temperatures. If the device displays a temperature of 36.4 °C (97.5 °F) or less with an LED backlight color of green or white, and the individual is exhibiting atypical symptoms or behaviors, contact your physician or health care professional.

DO NOT allow children to take their temperatures unattended.

Please consult your physician if you see symptoms such as unexplained irritability, vomiting, diarrhea, dehydration, changes in appetite or activity, seizure, muscle pain, shivering, stiff neck, pain when urinating, etc., even in the absence of fever.

Even in the absence of fever, those who exhibit a normal temperature may still need to receive medical attention. People who are on antibiotics, analgesics, or antipyretics should not be assessed solely on temperature readings to determine the severity of their illness.

DO NOT modify this equipment without the authorization of the manufacturer.

Why Braun No touch + touch?


Measurement in under 2 seconds

The innovative infrared technology allows measurement of forehead temperature in under 2 seconds whether you use the touch or no touch option.

Accurate and reliable


Due to the unique technology, the No touch + touch thermometer captures the heat naturally given off by the forehead to calculate body temperature value. Get the same professional accuracy whether touching your child's forehead or holding up to 2.5cm away.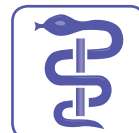

Easy to use

The No touch + touch thermometer is non-invasive. A measurement can be taken even while a child is sleeping.

The No touch + touch thermometer is safer to use on a child than a rectal thermometer and is easier to use than other methods.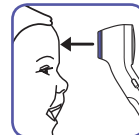

Safe and hygienic

No touch option helps minimize spreading of germs.

Completely safe for use on children and adults.

How does Braun No touch + touch work?

The No touch + touch thermometer measures infrared energy radiated from the skin at the center. This captured energy is collected through the lens and converted to an oral equivalent value.

The No touch + touch thermometer has been clinically tested and proven to be safe and accurate when used in accordance with its operating instruction manual.

Age Precision® colour-coded display

Clinical research shows the definition of fever changes as newborns grow into children and children grow into adults.¹

Select the appropriate age setting, take a temperature, and the display will glow green, yellow, or red to indicate whether the temperature is in a normal, fever, or high fever range, respectively.

The device is only intended for the measurement of human body temperature. The device also has a food/bath mode. It is not intended to diagnose or monitor a serious medical condition or disease.

¹ Herzog L., Phillips S. Addressing Concerns About Fever. Clinical Pediatrics. 2011; 50 (85): 383-390.

Temperature taking hints

It is important to know each individual's normal temperature when they are well. This is the only way to accurately diagnose a fever. Take multiple readings when healthy to determine normal temperature.

A child's normal temperature can be as high as 37.7 °C (99.9 °F) or as low as 36.1 °C (97.0 °F). Be sure to note this unit reads 0.5 °C (0.9 °F) lower than a rectal digital measurement.

An individual must be inside for 30 minutes before taking a measurement.

Note: The thermometer and individual should be in the same ambient temperature for at least 10 minutes.

ALWAYS hold the thermometer and the forehead steady when taking a reading. **DO NOT** move the thermometer until you hear the final beep. **DO NOT** take a measurement while or immediately after nursing a baby.

Individuals should not drink, eat, or be physically active before/while taking the measurement.

Remove hats and wait 10 minutes before taking a measurement.

Before taking a measurement, remove dirt or hair from the forehead area. Wait 10 minutes after cleaning before taking measurement.

ALWAYS take the temperature exactly as directed. Temperature results may vary if positioned in the wrong location.

For individuals measuring their own temperature, it is recommended to use the "touch" option instead of "no touch."

In the following situations it is recommended that three temperatures in the same location be taken and the highest one taken as the reading:

- Newborn infants in the first 100 days.
- Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.

- When the user is learning how to use the thermometer for the first time until he/she has familiarized himself/herself with the instrument and obtains consistent readings.


General precautions:

- Remove any sweat prior to measuring with a dry cloth.
- Avoid any cooling or warming cloths on the forehead for at least 30 minutes prior to measurement.
- **DO NOT** take temperature measurements over scar tissue, open sores, or abrasions.
- Keep the measurement sensor and lens clean. Avoid directly touching the sensor or lens.
- **ALWAYS** store the thermometer with the protective cap in place to prevent dirt and scratches from damaging the device.
- Make sure to remove the protective cap before taking a measurement and to put the cap back on after using the thermometer.

How to use your Braun No touch + touch

Note: The thermometer and individual should be in the same ambient temperature for at least 10 minutes.


1 Remove protective cap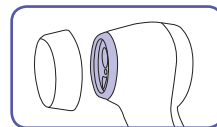

2 Power on

Press and release the power button once. Backlight will come on and the start-up sequence begins.

NOTE: Make sure to remove the protective scanner cap before taking a measurement.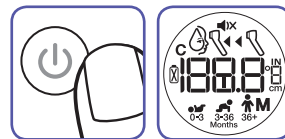

3 Select age range

Select age with Age Precision* button.

Device will start cycling through the age ranges, highlighting the 3 ranges you may select; 0-3 months (👶), 3-36 months (👧), or 36+ months (👤).

Press the Age Precision* button to toggle through the age ranges until you select the age appropriate for the individual.

3.1 To set a preferred age range: (Locking an age setting)

To save a preferred age range, so that the preferred age range will be automatically selected when the thermometer is turned ON, follow these steps:

A. Make sure the thermometer is OFF.

B. Press and hold the power button (🔘) and Age Precision button (⌚) for 4 seconds. The unit will turn on, displaying a yellow backlight and the 3 age range icons will flash.

C. Press the Age Precision (⌚) button. The age range selected will flash. Press the Age Precision button until your desired age range is flashing on the display.

D. Once the desired age range is selected, stop pressing the button and wait 3 seconds. The green backlight will illuminate, and the selected age range will turn solid. After 4 seconds, the thermometer will turn off.

E. The preferred age range will be automatically selected when the thermometer is turned ON.

Note: With a preferred age range, an alternate age range can be selected during each use, by pressing the Age Precision button when the age range icon appears, after the thermometer is turned ON.

F. To remove the preferred age range, open the battery door, remove the batteries, wait 30 seconds and re-insert the batteries.

IMPORTANT: To ensure accurate readings, you must always take a temperature reading in the age range that corresponds to the individual's age.

4 Position

Position thermometer on or up to 2.5 cm away from the center of the forehead, just between the eyebrows.

When the thermometer is ready and on or 2.5 cm away from the forehead a horizontal line of dashes ("---") will appear on the screen.




For No touch readings, the yellow guidance light will show you where you are aiming.

If the eyebrow area is covered with hair, sweat or dirt, please clean the area beforehand to improve the reading accuracy. It is important to hold the thermometer and the forehead steady during measurement. Movement will impact the temperature reading.

5 Take temperature

Three dashed lines mean the thermometer is ready to take a temperature. Press the temperature button. The display will show a dashed line animation while it takes a reading. Once the reading is complete a confirmation beep will be heard and the appropriate fever light color will be displayed on screen.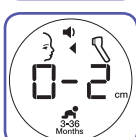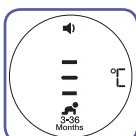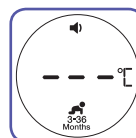

If the device is positioned too far away from the forehead, it will prompt you to move closer by displaying the characters "0-2 cm". Slowly move the device toward the forehead until the dashed line animation starts and a reading is displayed.

6 Read temperature

If temperature is in the normal range, a long beep will sound when the measurement is complete. If the temperature is in the fever range, ten short beeps will sound to indicate the measurement is complete.

Age range	Green Normal temperature	Yellow Fever	Red High Fever
👶 0* - 3 Months	$\geq 35.8 - \leq 37.4^{\circ}\text{C}$ ($\geq 96.4 - \leq 99.4^{\circ}\text{F}$)		$> 37.4^{\circ}\text{C}$ ($> 99.4^{\circ}\text{F}$)
👧 3 - 36 Months	$\geq 35.4 - \leq 37.6^{\circ}\text{C}$ ($\geq 95.7 - \leq 99.6^{\circ}\text{F}$)	$> 37.6 - \leq 38.5^{\circ}\text{C}$ ($> 99.6 - \leq 101.3^{\circ}\text{F}$)	$> 38.5^{\circ}\text{C}$ ($> 101.3^{\circ}\text{F}$)
👤 36 Months - adult	$\geq 35.4 - \leq 37.7^{\circ}\text{C}$ ($\geq 95.7 - \leq 99.9^{\circ}\text{F}$)	$> 37.7 - \leq 39.4^{\circ}\text{C}$ ($> 99.9 - \leq 103.0^{\circ}\text{F}$)	$> 39.4^{\circ}\text{C}$ ($> 103.0^{\circ}\text{F}$)


* 0 months is defined as having a gestational age of greater than or equal to 37 calendar weeks at birth.

7 To turn off

Press the power button to turn off. Device will also shut off automatically after 30 seconds of no use.

How to use the No touch setting to take a food/bath temperature


1 Remove protective cap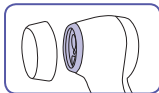

2 Power on


Press and release the power button once. Backlight will come on and the start-up sequence begins.

NOTE: Make sure to remove the protective scanner cap before taking a measurement.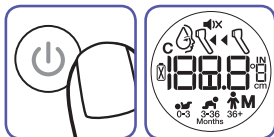

3 Select food & bath temperature mode


Press and release the food & bath temperature mode button. A beep will sound, the food & bath icon (🍲) will be displayed, the screen backlight will turn blue and three dashes will show on the screen indicating the thermometer is ready to take a temperature.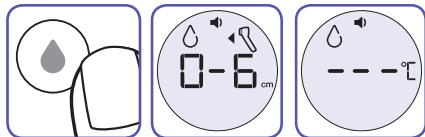

4 Position

Position device up to 6 cm away from the bath water or food.

Note: When measuring food or bath temperature the thermometer must be aimed directly at the food or bath water. **DO NOT** aim at sides of container holding the liquids or food. **DO NOT** touch liquid or food with the thermometer.

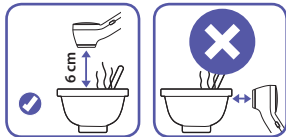
5 Take temperature

Press the temperature button (you can press and hold the button or press and release it). The display will show a dashed line animation while it takes a reading. Once the reading is complete, the display will show the temperature reading.



6 Read temperature

Remove the device and read the temperature.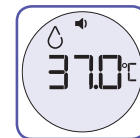

Changing the temperature scale

- 1 Open battery compartment and remove the batteries.
- 2 °C / °F switch is accessible in the battery compartment.
- 3 Slide switch to °C or °F to set preferred temperature scale.
- 4 Replace the batteries and close the battery door.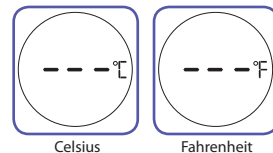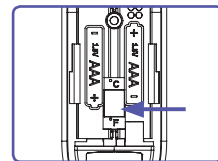


Celsius

Fahrenheit

Changing the sound mode



The Braun No touch + touch thermometer allows you to silence the beeps on the thermometer to avoid waking a sleeping child. To activate the silence feature, slide the switch to the silent mode position and take a temperature reading. An icon will appear on the screen to indicate that the thermometer is in silent mode.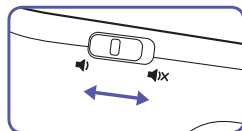

Calibration

The thermometer is initially calibrated at the time of manufacture. If this thermometer is used according to the use instructions, periodic re-adjustment is not required. If at any time you question the accuracy, please contact Consumer Relations.

Errors and troubleshooting

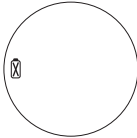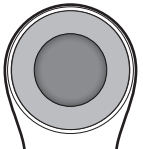Error message	Situation	Solution
	When 20% of battery life is left, the display flashes the low battery warning symbol; however, the device can still work until the battery life has 0% left.	Replace batteries.
	If the steady battery icon is the only symbol shown on the display, the device cannot work. The battery should be replaced immediately.	Replace batteries.
	This message displays when the measured temperature is lower than 34 °C (93.2 °F) or higher than 43 °C (109.4 °F) or when the room temperature is outside the operating range of 15 °C – 40 °C (59 °F – 104 °F).	Re-measure the temperature, carefully following the instructions in "How to use" section.
	System error – self-check display flashes continuously and will not be followed by the ready beep and the ready symbol. If error persists,	Wait 1 minute until the thermometer turns off automatically, then turn on again. ... reset the thermometer by removing the batteries and putting them back in. ... please contact Customer Service.
	Blank display. Thermometer does not have power.	Please check if the batteries have been loaded correctly. Also check polarity (<+> and <->) of batteries. Contact customer service if thermometer still does not function.

Care and cleaning

Use an alcohol swab or cotton swab moistened with alcohol (70% isopropyl) to clean the thermometer casing and the measuring probe. Ensure that no liquid enters the interior of the thermometer. Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the instrument in water or other cleaning liquids. Wait 10 minutes after cleaning before taking a temperature measurement. Make sure to replace the protective cap after use to prevent scratches or damage from occurring to the thermometer.

Never insert a sharp object into the scanner area or any other open surface on the thermometer.

Replacing the batteries

The No touch + touch thermometer comes with 2 AAA batteries. Replace with 2 new AAA batteries when the flashing battery symbol appears on the LCD screen. To change the batteries, slide open the battery cover and remove batteries. Replace the batteries being sure to align properly as indicated inside the battery compartment. Remove the battery from the product if it is not required for extended periods of time in order to avoid damage to the thermometer resulting from a leaking battery.

To protect the environment, dispose of empty batteries at appropriate collection sites according to national or local regulations.

Guarantee

Please read all instructions before attempting to use this device. Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant guarantee period. Any claim under the guarantee will not be valid without a proof of purchase.

Your device is guaranteed for Two Years (2 Years) from date of purchase.

This guarantee covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge.

The guarantee DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The guarantee becomes void if the device is opened, tampered with, or used with non-Braun branded parts or accessories, or if repairs are undertaken by unauthorised persons.

Accessories and consumables are excluded from any warranty.

For support requests, please visit www.hot-europe.com/support or find service contact information at the end of this owner's manual.

This Guarantee is applicable to Europe, Russia, Middle East and Africa only.

UK Only: This does not affect your consumer statutory rights.

The LOT and SN of your device are printed on the rating label in the battery compartment.

Manufacturing date is given by the LOT number. The first three digits represent the Julian date that the product was manufactured and the next two digits represent the last two numbers of the calendar year the product was manufactured.

Product specifications

Type:	No touch + touch (BNT400B)
Measuring range:	34 °C – 43 °C (93.2 °F – 109.4 °F)
Resolution:	0.1 °C (0.1 °F)
Laboratory Accuracy:	0.2 °C for 35 °C – 42 °C (0.4 °F for 95 °F – 107.6 °F) outside that range ± 0.3 °C/0.5 °F (Ambient Temperature: 15 °C to 40 °C (59 °F to 104 °F)) This thermometer displays a calculated oral equivalent estimate.
Display:	Liquid Crystal Display, 4 digits plus special icons
Acoustic:	Audio: Normal temp range = Green temp range: 1 long beep for 0.4 second duration Fever = Red or Yellow temp range: 10 short beeps for 0.2 second duration
Operating temperature:	15°C – 40 °C (59 °F – 104 °F)
Automatic Switch-off:	Approx. 30 seconds after last measurement has been taken
Weight:	100 gr (with batteries), 82.5 gr (w/o batteries)
Long term storage ranges	
Storage/transport temperature:	-25 °C – 55 °C (-13 °F – 131 °F)
Humidity:	15–95% non-condensing
Battery:	(2) AAA Batteries - at least 500 measurements
Pressure:	700-1060 hPa (0.7-1.06 atm)

This product conforms to the provisions of the European directives 93/42/EEC (medical device directive) and 2011/65/EU (ROHS).

Equipment with type BF applied parts

IP22

Protected against solid foreign objects of 12.5 mm diameter and greater. Protected against vertically falling water drops when the device is tilted up to 15 degrees.

+15°C
+40°C

Operating temperature

See instruction for use

-25°C
+55°C

Storage temperature

Internally Powered Equipment

Continuous Operation

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC.

For detailed description of EMC requirements please refer to www.hot-europe.com/support.

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Remove the battery from the instrument if it is not required for extended periods of time in order to avoid damage to the thermometer resulting from a leaking battery.

This product contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in the household waste, but take it to appropriate local collection points.

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